

Blue to the sky. Orange to the thigh.

How to use EpiPen® and EpiPen® Jr (epinephrine) Auto-injectors.

Remove the EpiPen® Auto-Injector from the carrier tube and follow these 2 simple steps:



- Hold firmly with orange tip pointing downward.
- **Remove blue safety cap by pulling straight up. Do not bend or twist.**



- **Swing and push orange tip firmly into mid-outer thigh until you hear a 'click'.**
- Hold on thigh for several seconds.



Built-in needle protection

- After injection, the orange cover automatically extends to ensure the needle is never exposed.



After using EpiPen®, you must seek immediate medical attention or go to the emergency room. For the next 48 hours, you must stay close to a healthcare facility or be able to call 911.


For more information visit the consumer site EpiPen.ca.

EpiPen® and EpiPen® Jr (epinephrine) Auto-Injectors are indicated for the emergency treatment of anaphylactic reactions in patients who are determined to be at increased risk for anaphylaxis, including individuals with a history of anaphylactic reactions. Selection of the appropriate dosage strength is determined according to patient body weight.

EpiPen® and EpiPen® Jr Auto-Injectors are designed as emergency supportive therapy only. They are not a replacement for subsequent medical or hospital care. After administration, patients should seek medical attention immediately or go to the emergency room. For the next 48 hours, patients must stay within close proximity to a healthcare facility or where they can call 911. To ensure this product is right for you, always read and follow the label. Please consult the Consumer Information leaflet in your product package for complete dosage and administration instructions.



© 2015 Pfizer Canada Inc., Kirkland, Quebec H9J 2M5 • Toll free: 1-877-EPIPEN1 (1-877-374-7361)
EpiPen®, EpiPen® Jr are registered trademarks of Mylan, Inc. licensed exclusively to its wholly-owned affiliate,
Mylan Specialty, L.P.; sub-licensee, Pfizer Canada Inc., Kirkland, Quebec H9J 2M5
D000050559

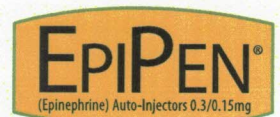

Trusted for over 25 years.